

ESSENTIALLY BETTER



Vida

Clear design and high practicality

Vida

Dependable and practical

The Vida hospital bed combines modern design with great reliability and practicality. Its clean lines and clear design make it easy not only to handle but also to reprocess between bed occupation. Due to its large adjustment range this bed can be individually set to a comfortable entry height to allow the patient to get in and out of bed. The lowest position enhances patient safety and helps to prevent fall-related injuries. At the same time, care staff are always able to position the patient at an ergonomic working height. The integrated "Auto Contour" function allows simultaneous adjustment of the backrest and thigh rest. This provides greater comfort for the patient and also prevents the patient from sliding towards the foot end when the bed moves into a sitting position. The stable design provides for a safe working load of up to 250 kg. The Vida hospital bed is also ideally suited to heavier patients.

The motorised hospital bed

Contents

4 Control units

6 Features

8 Safety sides

9 Headboards, footboards and decors

10 Equipment and technical data

11 The Stiegemeyer Group



Vida with MultiFlex safety side

Safety and mobility

- Split safety side for getting in and out of bed without any obstructions
- Middle post helps mobilise the patient
- Protective height 41.5 cm



Vida with 3/4 safety side

Easy to operate

- Easy to lower the height
- No additional space is required beside the bed
- Protective height 39 cm



Control units

Combinations to suit

The Vida can be handled easily and intuitively at any time with its control units. Combinations of a handset and control box are available to cover a wide range of needs.

The ergonomically shaped **patient handset** offers easy one-hand operation. Two different versions are available: with direct locking function or with locking function via the control box.

With the **control box**, care staff have access to all the usual adjustment options for the mattress base, plus, for example, the CPR

function and the Trendelenburg position. They can also lock any of the standard handset functions individually to prevent the patient from using them.

The control box is integrated in the linen holder in the bed. There, it is protected but still easily accessible to care staff. In order to reach the control box even more quickly during the day-to-day routine it can be hooked onto the footboard of the bed.



▲ With the control box, all the adjustments of the mattress base can be operated and functions can be locked for the patient.




▲ An optional handset is available that can be combined with the control box.



▲ The handset is lockable as standard and allows the Trendelenburg position to be set quickly in an emergency.

External dimensions/height adjustment range:

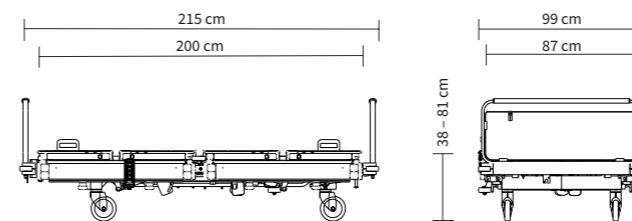

High safe working load 250 kg	Large height adjustment range 38 to 81 cm	Trendelenburg position	Bed extension
----------------------------------	--	------------------------	---------------

Equipment

Relief in many everyday situations

The Vida can be delivered with numerous optional features. These useful features significantly facilitate the daily care routine for care staff. The Vida can be fitted with an **under bed light**, for example. This is a useful orientation aid for patients at night, without disturbing the patient in the next bed, and helps to prevent falls. Beds fitted with an under bed light can also incorporate an acous-

tic **brake signal**. This indicates that the brakes are not applied although the power cable from the bed is still plugged in. The Vida can be supplied with a **hygiene mattress base**. This mattress base is made of tough polypropylene. The plastic elements are specially formed to make cleaning easier.



▲ The mattress base can also be equipped with a radiolucent HPL backrest as an optional feature.

▲ The optional x-ray cassette holder under the backrest makes the Vida suitable for use in many different hospital units.



▲ The mattress base sections are extremely light and can be removed separately. This simplifies manual reprocessing.



▲ The bed extension allows the mattress base to be extended to a comfortable length for tall patients (optional feature).



▲ Integral castors with central braking.



▲ The supply rail offers flexible options for attaching accessories to the bed (optional).



▲ The slide-out linen holder is a practical aid when changing bed linen or treating the patient.



▲ A spirit level for the angle of tilt can be integrated into the wall deflection rollers at the foot end.



▲ The angle indicator enables the backrest to be tilted precisely to the medically required position (optional).



▲ The emergency lever allows the backrest to be lowered quickly and safely in one easy movement.



▲ The ¾ safety side with a height of 39 cm covers a large proportion of the mattress base and affords the patient an uninterrupted view.



▲ The MultiFlex safety side has a protection height of 41.5 cm.



▲ The ¾ safety side can be effortlessly swivelled away in just a few easy steps.



▲ The fold-down supports of the MultiFlex safety sides also serve as a mobilisation aid.



▲ The Classic headboard/footboard has a robust steel frame and a reinforced handle bar for transporting the bed.



▲ The Intercontinental headboard/footboard is made of plastic and is particularly easy to clean.

▼ Both models can be quickly and easily removed.




Headboards, footboards and decors

Versatile combinations

Two versions of head and footboard are available for the Vida: **Classic** and **Intercontinental**. The head and footboards can be quickly and easily removed for treatment or cleaning purposes. In an emergency, this also allows fast access to the patient.

* These decors can be selected for the Intercontinental head and footboard.

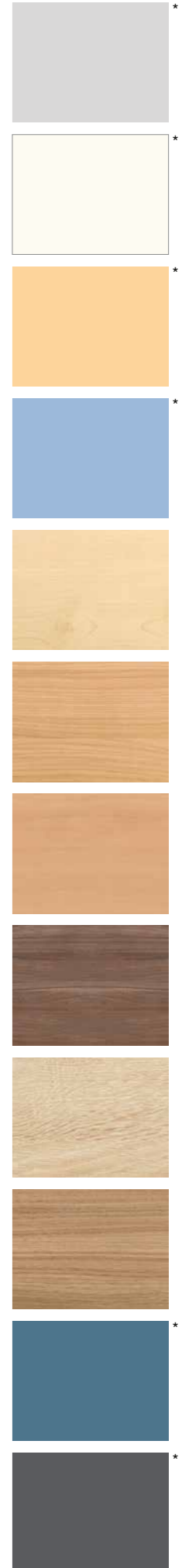

Safety sides

Individual protection

The Vida is supplied with a ¾ **safety side** as standard. The sturdy bars provide safe protection while affording the patient an uninterrupted view into the room or through the window. The ¾ safety side can be easily and quietly swivelled away and raised again.

The optional **MultiFlex** split safety side combines excellent protection with the freedom to leave the bed at any time without any obstruction.

ESSENTIALLY BETTER

MultiFlex safety sides

The MultiFlex split safety side adapts to the needs of the patient at all times. When the safety side is raised at the head end only, the patient can leave the bed without obstruction, yet is still protected from falling out of bed.

- The fold-down supports of the safety side can also be used as a mobilisation aid
- MultiFlex safety sides do not require any additional space along the sides of the bed
- Easy handling

Equipment and technical data

Equipment and options	
Safety sides	
¾ safety side	✓
MultiFlex safety sides	○
Head and footboard design	
Classic	✓
Intercontinental	○
Base colours	
Crystal white NCS S0500N	✓
Argentum similar to RAL 9006	○
Mattress bases	
PP hygiene mattress base	✓
PP hygiene mattress base for belt system	○
PP hygiene mattress base, with radiolucent backrest	○
PP hygiene mattress base with radiolucent backrest for belt system	○
Castors	
Integral castor Ø 150 mm	✓
Conductive castor coating	
1 castor	✓
2 castors	○
4 castors	○
Directional castor position	
Head end, right	✓
Head end, left	○
Foot end, right	○
Foot end, left	○
Control units	
Control box	○
Handset with locking function via control box	○
Handset with direct locking function	✓

Additional features	
Under bed light	○
Brake signal ¹⁾	○
Standard sliding rail	○
Supply rail	○
Backrest angle display	○
Spirit level for angle of tilt	○
Linen Holder	✓
Rechargeable battery	○
X-ray cassette holder	○
Lower leg rest extension	○
Bed extension	✓

Technical data	
Dimensions and weights	
Overall dimensions	99 x 215 cm
Mattress base (mattress dimensions)	87 x 200 cm
Bed extension	28 cm
Mattress base division	82 – 20 – 36 – 53 cm
Backrest length compensation	9 cm
Safe working load	250 kg
Maximum weight of patient	240 kg
Ground clearance of chassis	approx. 15.5 cm
Total weight	149 kg
Protection height of ¾ safety sides	39 cm
Protection height of MultiFlex safety sides	41.5 cm
Adjustment ranges	
Tilting to Trendelenburg position	approx. 16°
Tilting to reverse-Trendelenburg position	approx. 16°
Mattress base height	38 – 81 cm
Backrest angle ²⁾	70°
Thigh rest angle	43°



Custodians of values Shapers of the future

STIEGELMEYERGROUP

Custodians of values

Our company was founded in Herford in 1900 and is still a family-run enterprise today. Respect and a sense of responsibility towards our customers, employees and partners form the basis for our day-to-day work. Based on these values and our many years of experience, we offer our customers all over the world high-quality products, solutions and services to meet the needs of the care and healthcare sectors.

Shapers of the future

In order to cope with rapid changes in the care and healthcare sectors, we develop all our products ourselves. Creating innovations and visions of what care will be like in future are the goals we pursue. We maintain regular contact with our customers in order to support them in their daily tasks. Implementing tomorrow's trends today is a successful tradition of ours.

Our expertise

Hospital

Nursing home

Homecare

Lifestyle beds

Furniture systems

Service



The Stiegelmeier-Group is your international partner for hospitals and care facilities and, with the Burmeier brand, also for home care. We support our customers with practical products, solutions and services, which we constantly develop further.

- Founded in 1900 and still family-run today
- Employs 1,100 members of staff worldwide
- Production of all core products in our own factories
- Product development and manufacture in Germany
- Services, training and process optimisation at the customer's site
- International certification and export to more than 60 countries
- Sales companies in Belgium, Finland, France, the Netherlands, Poland and South Africa

Our brands





STIEGELMEYERGROUP

Hauptsitz / Headquarters

Deutschland / Germany
Stieglmeyer GmbH & Co. KG
Ackerstraße 42
32051 Herford
Phone +49 (0) 5221 185 - 0
Fax +49 (0) 5221 185 - 252
info@stieglmeyer.com
www.stieglmeyer.com

France
Stieglmeyer SAS
8 Allée des Chevreuils
69380 Lissieu
Phone +33 (0) 4 7238 - 5336
Fax +33 (0) 4 7238 - 5481
contact@stieglmeyer.fr
www.stieglmeyer.fr

South Africa
Stieglmeyer Africa (PTY) Ltd
The Estuaries · Century City
Cape Town, 7441
Phone +27 (0) 21 555 - 1415
Fax +27 (0) 21 555 - 0639
info.southafrica@stieglmeyer.com
www.stieglmeyer.co.za

Nederland / Netherlands
Stieglmeyer BV
Peppelenbos 7
6662 WB Elst
Phone +31 (0) 345 581 - 881
info@stieglmeyer.nl
www.stieglmeyer.nl

Suomi / Finland
Stieglmeyer Oy
Ahjokatu 4 A
15800 Lahti
Phone +358 (0) 3 7536 - 320
Fax +358 (0) 3 7536 - 330
stieglmeyer@stieglmeyer.fi
www.stieglmeyer.fi

Polska / Poland
Stieglmeyer Sp. z o.o.
86-212 Stolno
Grubno 63
Phone +48 56 6771 - 400
Fax +48 56 6771 - 410
info.stolno@stieglmeyer.pl
www.stieglmeyer.pl



Finden Sie Ihren Ansprechpartner auf unserer Website:
Find your local partner on our website:
www.stieglmeyer.com/partner

stieglmeyer.com